**BAUERFEIND AG**

Triebeser Straße 16
07937 Zeulenroda-Triebes
Germany
P +49 (0) 36628-66-40 00
F +49 (0) 36628-66-44 99
info@bauerfeind.com

GenuPoint

Patellasehnenbandage


Patella tendon strap

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
Vendaje para el tendón patelar


1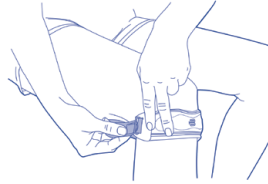

2

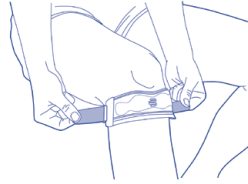3

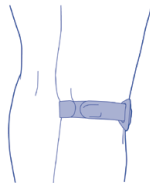4

(de) deutsch..... 3	(sr) srpski.....32
(en) english..... 5	(sl) slovenski.....33
(fr) français..... 6	(ro) romanian.....35
(nl) nederlands..... 8	(el) ελληνικά.....37
(it) italiano.....10	(tr) türkçe.....39
(es) español.....12	(ru) русский.....41
(pt) português.....14	(et) eesti.....43
(sv) svenska.....16	(lv) latviešu.....44
(no) norsk.....18	(li) lietuvių.....46
(fi) suomi.....20	(uk) українська мова.....48
(da) dansk.....21	(he) עברית.....50
(pl) polski.....23	(ja) 日本語.....53
(cs) česky.....25	(ko) 한국어.....54
(sk) slovensky.....26	(ar) عربي.....57
(hu) magyar.....28	(zh) 简体中文.....60
(hr) hrvatski.....30	

(de) deutsch

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en english

Dear Customer,

Thank you for choosing a product from Bauerfeind. GenuPoint is a high-quality medical product.

Please read through these **instructions for use** carefully and if you have any questions, contact your physician, medical retailer or our Technical Service department.

Intended use

GenuPoint is a product for treating anterior knee pain (patellar tendinitis or jumper's knee). The support contains a massage pad with four pressure points that guide and relieve the affected tendon at the knee. The central pressure points guide and support the tendon, while the side pressure points improve the muscular coordination of the knee joint. The pressure exerted by the support can be set using a freely adjustable Velcro strap. GenuPoint can be worn on the right or left leg.

If the product is not used properly, all warranty rights will be rendered null and void.

Indications

- Pain in the kneecap area and directly below the knee (patellar chondropathy, patellar tendinitis/jumper's knee)

Risks of using this product

⚠ Caution*

- GenuPoint is a prescribable product. In case you have received it by prescription from your doctor or physiotherapist, the use and duration of use must be agreed with her/him before utilizing the product. In any case, instruction is required by the specialist who provided you with the product. GenuPoint should only be applied in accordance with the

instructions contained in these usage directions and for the listed areas of application.

- Only a trained specialist¹ may carry out the (initial) fitting of the support and provide instruction.
- This product is intended to be fitted only once and to one patient.
- Product liability cannot be assumed in the event of inappropriate use.
- Do not alter the product in any way. Failure to comply with this requirement may adversely affect product performance, thereby canceling any product liability.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your physician without delay.
- Combining the support with other products (e.g. compression stockings) must be discussed beforehand with your physician.
- The product's unique effect is produced primarily during physical activity. The product should be removed during prolonged rest breaks (e.g. sleeping, continuous sitting).
- Avoid contact with ointments, lotions or any substances containing grease and acids.
- For information regarding the options available for returning the packaging to ensure it is correctly recycled, please observe the instructions from the sales outlet where you purchased this product.

No side effects that affect the entire body have been reported to date. Correct usage/fitting is assumed. Any medical aids – supports and orthoses² – applied externally to the body can, if tightened excessively, lead to local pressure symptoms or, rarely, constrict the underlying blood vessels or nerves.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

- Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Varicose veins (varicosis).
- Impaired sensation and circulatory disorders of the legs (e.g. diabetes mellitus).
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid.

Application instructions

Putting on the GenuPoint

The GenuPoint can be worn on the right or left leg. Before putting it on, please loosen both Velcro closures. Open one side completely by pulling the end out of the tab.

Put the GenuPoint on in the center of the kneecap with the semicircular cut-out edge at the top **1**

Then thread the Velcro strap back through **2**

Now pull both strap ends at the same time with equal force to prevent the support from twisting. Make sure that the support is sitting correctly and is level with the patellar tendon **3**. You can adjust the pressure on the patellar tendon to suit you, but it must not be uncomfortable.

Now fasten both Velcro ends to the strap **4**.

Depending on your needs and activity level, you can readjust the individual pressure on the patellar tendon.

Taking off the GenuPoint

To take off the support, first loosen both Velcro closures and then remove one side fully from the guide tab. For storage, both strap ends should be fastened again to avoid damage.

Cleaning instructions

Note: Never expose the GenuPoint to direct heat (e.g. heaters, sunlight, in your car, etc.)! This may damage the material, which can impair the effectiveness of the GenuPoint.

The sewn-in label of the GenuPoint, which contains information on the product name, size, manufacturer, washing instructions and CE mark, is located on the strap of the product.

Fasten both ends of the Velcro strap before washing to avoid damage. Wash the GenuPoint separately by hand using a mild detergent.

If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.

Notes on reuse

The GenuPoint is intended for your treatment only. It has been adjusted specifically for you. For this reason, do not pass the GenuPoint on to other people.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made

under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disposal

You can dispose of the product in accordance with national legal requirements.

Specialists¹

General instructions

- Check the optimal and individual fit of the GenuPoint after the first fitting. Make any corrections if necessary.
- Practice the correct procedure for putting on the support with the patient.
- No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect product performance, thereby voiding any product liability.
- The GenuPoint is intended for the treatment of one patient only.

Version: 2016-09

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

² Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control or correct the limbs or the torso

* Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

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Bauerfeind USA, Inc.